

Unosorb Filter



Features

The UNOSORB Anesthetic* Gas Filter consists of small grain activated charcoal for optimal adsorbtion of aneasthetic gasses

- * Isofluran, Halothane or Sevofluran.
- The UNOSORB filter is suitable for Animal Anaesthesia gas removal
- A disposable canister, containing activated charcoal
- The UNOSORB canister can be connected directly to the scavenging tubing (f.i. UNO Gas Exhaust Unit).
- The charcoal absorbs organic anaesthetic gases e.g. Sevoflurane, Isoflurhane and Halthane.
- The weight of the canister must be monitored so that it can be replaced before it becomes saturated, once saturated any waste gases will simply be exhausted into the working environment.

Ordering number

Ordering number	Product	Weight	Storage Capacity
180000140	UNO Sorb Filter	approx. 200gr	approx. 1.200 gr