



**CARFIL QUALITY**  
YOUR SUPPLIER FOR ALL LABORATORY NEEDS

## Vein Pick



## Hockey Stick Catheter Introducer

This simple vein pick catheter introducer makes the introduction of catheters into small vessels simple and easy. It holds the vessel open during the cut-down catheter insertion procedure. The tapered tip of the vein pick is inserted into an incision that is made with a micro-scissors or needle. The catheter then slides easily into the vessel by placing the catheter tip in the grooved underside of the vein pick. The vein pick is then removed and the catheter advanced to the desired position.



**PAVAN SERVICE BVBA**

Beyntellus 3 - B-2360 Oud-Turnhout - Tel. +32 (0)14 45 13 10 - [admin@carfil.be](mailto:admin@carfil.be) - BTW BE 0420.681.872

**[www.carfil.be](http://www.carfil.be)**